



Products » Healthy veins » [Donning with medi Butler](#)

Donning with medi Butler

Instruction for use

Step 1




Lay the compression stocking in the stirrup with the toe of the stocking facing forwards. Turn the upper border of the stocking inside out and over the frame.

Step 2



Slide the stocking down over the stirrup until the heel appears at the edge of the stirrup.

Step 3

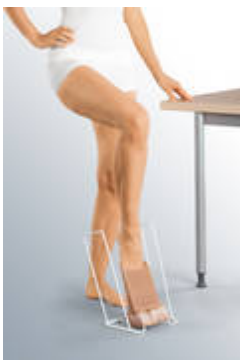

Now place the medi butler on the floor. Slip your foot, toes first, into the toe of the stocking. It is recommended that you lean against something firm (table, wall) for support.

Step 4



Now step into the rigid stand. Please check that the toe and the heel are correctly positioned.

Step 5



By pulling alternately on the handles you can now slide the stocking up towards your knee.

Step 6



When the stocking is just below the knee, tilt the medi butler slightly backwards and remove it from the stocking.

Step 7



To finish, smooth the compression stocking upwards evenly. Please make sure you ease the stocking up bit by bit without gathering it up or pulling on the top.

medi. ich fühl mich besser.